



Choose eakin dot[®] 2-piece: for a peristomal wound

4 reasons to use:

1. Choosing the correct appliance is an important aspect in a person's overall recovery after ostomy surgery.¹
2. The goal of both nurse and patient is to achieve a predictable, reliable wear time.²
3. Peristomal skin irritation is a very common early complication when managing a stoma.
4. Peristomal skin problems can be corrected with the appropriate pouching system.^{3,4}

Case study submitted by:


Jacqueline Cornelissen
Stoma care nurse, Canisius Wilhelmina
Ziekenhuis Nijmegen, Netherlands

References:

1. Williams J, Gwillian B, Sutherland N, Matten J, Hemmingway J, Ilsey H et al (2010) Evaluating skin problems in people with stomas. BJN 19(17): s6-15 2. Wound, Ostomy, Continence Nurses Society(WOCN). (2013) Colostomy and Ileostomy: Products and tips. Mt. Laurel, NJ: WOC 3. Wound, Ostomy, Continence Nurses Society(WOCN). (2016) Peristomal skin complications: Best practice for clinicians. 4. Hooper J (2018) Ostomy: Choosing the Right Appliance. Wound Care Education Institute www.woundcentral.com


Patient History:

- 69 year old man with a history of rectal carcinoma treated with chemo and radiotherapy and formation of permanent colostomy
- Independent with stoma care
- Discharged home using 1-piece drainable pouch changed daily

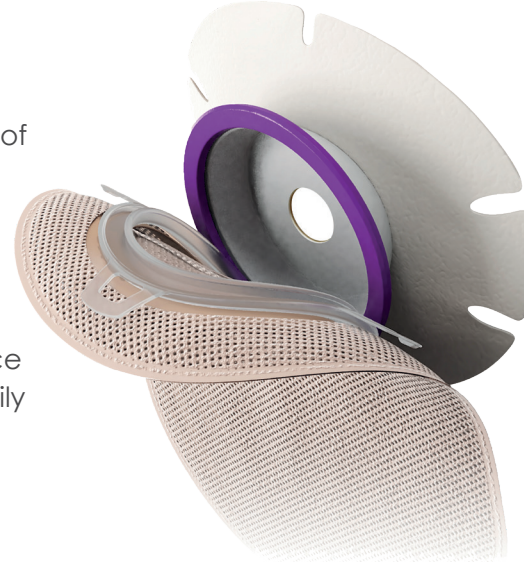

Problems:

- Painful, moist wound under stoma
- Frequent leakages

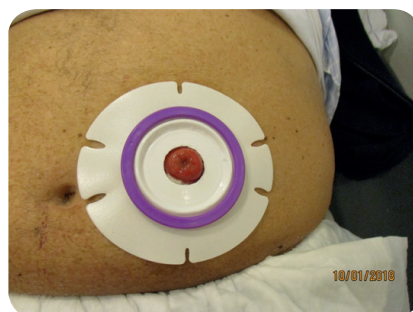

Nurse recommends eakin dot[®] 2-piece:

- eakin dot[®] 2-piece reduces trauma of frequent changes. Base plate changed twice per week
- Nurse also recommends using alginate dressing for 1 week to absorb moisture from wound



Nurse review:

- After **1 week** the wound is healing well and alginate dressing is discontinued
- eakin Cohesive SLIMS[®] added to the stoma care routine to help manage the moisture from the wound
- After **2 weeks**, the wound is completely healed and use of eakin dot[®] 2-piece is continued

Outcome:

- *skinsmart*[™] hydrocolloid for longer-life wear resulted in a wear time of 3-4 days
- The nurse and patient found eakin dot[®] 2-piece to be easy to use as well as highly effective in the treatment of complex skin problems and in alleviating the effects of skin trauma



A SMALL PART
OF A BIGGER LIFE

Clinical evidence for choosing eakin dot[®] 2-piece:

- Improves peristomal skin problems
- Ease of use
- *skinsmart*[™] hydrocolloid for longer-life wear
- Meets the individual needs of ostomates