



Choose eakin dot® 2-piece for peristomal skin problems

4 reasons to use:

1. Choosing the correct appliance is an important aspect in a person's overall recovery after ostomy surgery.¹
2. The goal of both nurse and patient is to achieve a predictable, reliable wear time.²
3. Peristomal skin irritation is a very common early complication when managing a stoma.
4. Peristomal skin problems can be corrected with the appropriate pouching system.^{3,4}

Case study submitted by:


Jozé Bensen & Romana Vrugteveen Stoma and wound care nurse. Röpcke Zweers Ziekenhuis/ Saxenburgh Groep

References:

1. Williams J, Gwilliam B, Sutherland N, Matten J, Hemmingway J, Ilsey H et al (2010) Evaluating skin problems in people with stomas. BJN 19(17): s6-15 2. Wound, Ostomy, Continence Nurses Society(WOCN). (2013) Colostomy and Ileostomy: Products and tips. Mt. Laurel, NJ: WOC 3. Wound, Ostomy, Continence Nurses Society(WOCN). (2016) Peristomal skin complications: Best practice for clinicians. 4. Hooper J (2018) Ostomy: Choosing the Right Appliance. Wound Care Education Institute www.woundcentral.com

Patient History:

- Mrs RJ had a history of diverticular disease and benign polyps
- Hartmann's resection with end colostomy in 2017
- Requires assistance from carers due to CVA (stroke) in 2011
- She cannot talk and has difficulty with understanding her situation
- Discharged from hospital using a 2-piece closed pouch

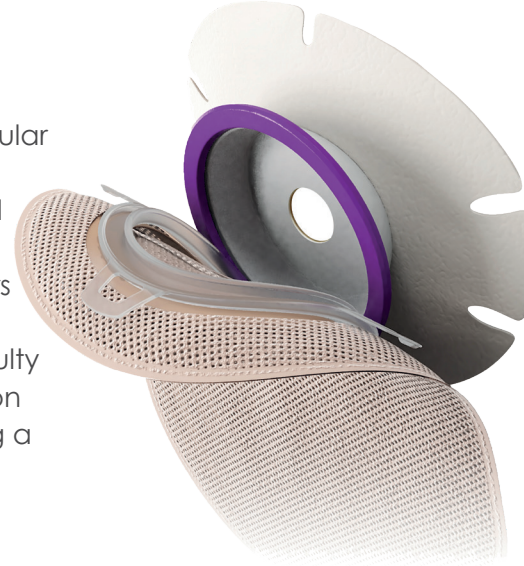

Problems:

- Developed periods of peristomal skin excoriation 9 months after surgery which has resulted in leakages

Nurse review:

- Peristomal skin excoriated
- Itchy skin – treated with anti-fungal cream
- Leakages
- Alternating output especially when on antibiotics for a urinary tract infection

Nurse recommends eakin dot® 2-piece:

- *skinsmart*™ hydrocolloid for longer-life wear extends wear time and adheres well to the skin, even with loose output
- eakin dot® 2-piece is easy for the care giver to use and restores confidence

Outcome:

- Peristomal skin improved
- No leakages
- Itchy skin resolved
- Feels secure

“Base plate of eakin dot® feels comfortable, easy to use and apply. Patient feels secure again.”



A SMALL PART
OF A BIGGER LIFE

Clinical evidence for choosing eakin dot® 2-piece:

- Improves peristomal skin problems
- Ease of use
- *skinsmart*™ hydrocolloid for longer-life wear
- Meets the individual needs of ostomates