

eakin dot®

80mm Ring Size

Choose eakin dot® 2-piece for challenges with a parastomal hernia

4 reasons to use:

1. Choosing the correct appliance has been identified as an important aspect in a person's overall recovery after ostomy surgery.¹
2. The goal of both nurse and patient is to achieve a predictable, reliable wear time.²
3. A parastomal hernia is a common complication following stoma forming surgery.
4. The presence of a parastomal hernia can make stoma management a challenge for both the nurse and the patient as it can contribute to reduced wear time and leakages.^{3,4}

Case study submitted by:

Emma Ludlow, STN, Counties, New Zealand

References:



1. Williams J, Gwillian B, Sutherland N, Matten J, Hemmingway J, Ilsey H et al (2010) Evaluating skin problems in people with stomas. BJN 19(17): s6-15 2. Wound, Ostomy, Continence Nurses Society(WOCN). (2013) Colostomy and Ileostomy: Products and tips. Mt. Laurel, NJ: WOCN 3. Osborne W, North J, Williams J (2018) Using a risk assessment tool for parastomal hernia prevention. British Journal of Nursing 27(5):15-19 4. Hooper J (2018) Ostomy: Choosing the Right Appliance. Wound Care Education Institute www.woundcentral.com

Patient History:

- 79 year old gentleman
- Permanent colostomy formed in 2003 for rectal cancer
- Large parastomal hernia (repaired 3 times)
- Due to cardiac history not fit for further surgery
- Tried multiple 1 and 2-piece flat and convex products over 15 years

Problems:

- Intermittent blockages high volume explosive output every 2 days
- Frequent leakages within 4 hours of application due to bending and high output
- Loss of confidence due to lack of appliance security

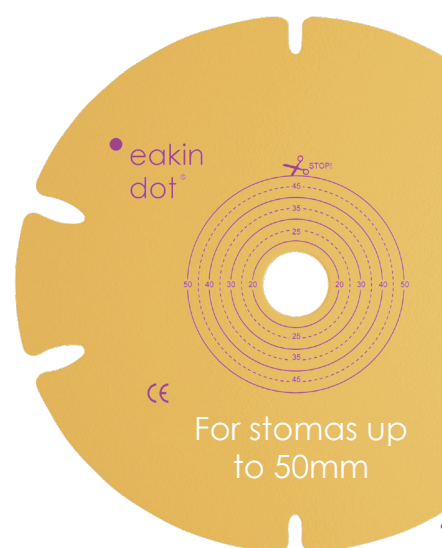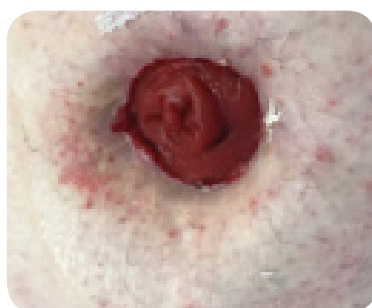

"eakin dot® wins hands down. The best by a mile. What more could I ask for?"

Nurse review:

- Stoma healthy, minimal protrusion, pulled below skin level due to hernia, sited in deep trough
- Multiple scars above and lateral to stoma
- Currently using a 2-piece convex with an eakin cohesive® seal
- Willing to try eakin dot® 2-piece system flat system with an eakin cohesive® seal

Nurse recommends

eakin dot® 2-piece 80mm ring:

- The large surface area cuts up to 50mm - ideal for a large hernia
- skinsmart™ hydrocolloid for longer-life wear offers instant tack and extended wear time

Outcome:

- skinsmart™ hydrocolloid for longer-life wear resulted in a wear time of 7 days
- Secure closure prevented pouch from falling off when bending
- Foam backing provided superior comfort
- Low, discrete profile was an added bonus

Clinical evidence for choosing eakin dot® 2-piece:

- Improves wear time
- Secure and reliable
- skinsmart™ hydrocolloid for longer-life wear
- Meets the needs of ostomates