

eakin dot®

2-piece flat system
60mm ring size

Choose
eakin dot®
2-piece
for a high
output stoma

4 reasons to use:

1. Increased patient security and comfort^{1,2}
2. Skin friendly *skinsmart*™ hydrocolloid for longer-life wear^{1,2}
3. Significant improvement in peristomal skin condition^{1,2}
4. Reduction in nursing care resources²

Case study submitted by:
Marieke Muller, Stoma Nurse,
Tergooi Hospital, Hilversum,
The Netherlands

1. eakin dot® 2-piece user evaluations. Data on file. 2017-2019
2. eakin dot® 2-piece nurse case study. Data on file. 2019

eakin
dot®

A SMALL PART
OF A BIGGER LIFE

Patient History:

- **Jan 2019:** Emergency surgery (open right hemicolectomy with an end ileostomy) due to bowel obstruction caused by tumour
- High output stoma
- **Feb 2019:** Discharged home on a 2-piece adhesive high output pouch
- Skin healthy. Changing baseplate 2 x per week and pouch 1 x per day
- Managing his stoma care independently



10 Apr 2019: Prior to using eakin dot® 2-piece flat system

Problems/ Nurse review:

- **3 Apr 2019:** Lots of residue and red skin
- Emptying 5-7 times per day
- Baseplate wet
- Reluctant to switch products as happy with outlet on current pouch
- Advised to change baseplate 3 x per week, pouch 1 x a day. Treat skin with barrier cream and remove residue with an eakin **release**™ adhesive remover wipe. Evaluation in one week
- **10 April 2019:** Skin is red and itchy

Nurse recommends eakin dot® 2-piece flat system:

- Recommended to switch to eakin dot® 2-piece flat system, precut 27.5mm baseplate and maxi pouch and stop using barrier cream
- Baseplate to be changed 2 x per week, bag 1 x a day
- Evaluation again in 2 weeks



23 Apr 2019: Significant improvement in peristomal skin after 2 weeks on eakin dot® 2-piece flat system

Outcome:

- **23 Apr 2019:** No leakages experienced
- Improved peristomal skin condition
- **21 May 2019:** Improved feeling of comfort and condition of peristomal skin
- Patient wants to continue use of eakin dot® 2-piece flat system

Clinical evidence for choosing eakin dot® 2-piece flat system:

- *skinsmart*™ hydrocolloid for longer-life wear is clinically proven to offer greater security, pouch wear time and skin-friendliness compared to other 2-piece hydrocolloids tested¹