

eakin dot®

2-piece flat system
80mm ring size

Choose
eakin dot®
2-piece
for sensitive skin

3 reasons to use:

1. Increased patient security, confidence and comfort^{1, 2}
2. Skin friendly *skinsmart*™ hydrocolloid for longer-life wear^{1, 2}
3. Significant improvement in peristomal skin condition^{1, 2}

Case study submitted by:
Marieke Muller, Stoma Nurse,
Tergooi Hospital, Hilversum,
The Netherlands

1. eakin dot® 2-piece user evaluations. Data on file, 2017-2019
2. eakin dot® 2-piece nurse case study. Data on file, 2019

Patient History:

- **13 Mar 2019:** Acute rectosigmoid resection with end colostomy due to sigmoid perforation (tumour)
- Very large post-op stoma
- Patient discharged to nursing home on an adhesive 2-piece system with closed pouch



Problems/ Nurse review:

- **29 Mar 2019:** Attended hospital - **hypersensitive severely excoriated skin.** Evidence of **skin stripping** and **blisters** requiring adhesive remover to remove baseplate
- Stoma reduced in size - changed to smaller 2-piece system
- Blisters treated with powder. Flange extender used around edge of baseplate
- Discharged to nursing home. Advised to change baseplate 3 x per week and closed pouch 1 x per day. Revisit in 1 week
- **2 Apr 2019:** **Painful broken skin** with blisters. **Leakages** under baseplate at left and right sides. Baseplate worn for 4 days. Pouch changed once per day



Nurse recommends eakin dot® 2-piece:

- **2 Apr 2019:** Attended hospital - recommended to **change** to eakin dot® flat 80mm ring size with eakin Cohesive StomaWrap™ and flange extender on right side. Drainable pouch used due to loose output. Discharged to nursing home with recommendation to **change baseplate 2 x per week** and continue with daily pouch changes
- **5 Apr 2019:** Baseplate changed in nursing home with daily pouch changes and emptying throughout the day



Outcome:

- **8th Apr 2019:** **Significant improvement in peristomal skin**
- **9th Apr 2019:** Hospital appointment. Gentleman able to **change baseplate himself** using adhesive remover spray and reported **reduced pain**
- **12th Apr 2019:** Nursing home changed flange
- **15th Apr 2019:** Returned to using closed pouch, changing 1-2 times/day. **No leakages, output is thicker and skin has healed within 2 weeks of switching to eakin dot® 2-piece flat system**



Clinical evidence for choosing eakin dot® 2-piece flat system:

- *skinsmart*™ hydrocolloid for longer-life wear is clinically proven to offer greater security, pouch wear time and skin-friendliness compared to other 2-piece hydrocolloids tested.¹

eakin dot®

A SMALL PART
OF A BIGGER LIFE