1

2

3

4

5

6

eakin **Cohesive**® seals are moisture absorbing skin protecting seals which are intended to protect peristomal or periwound skin from harmful attack by aggressive body fluids. High tackiness on both sides ensures that any subsequent appliance is held securely in position on the body.

Intended use

- An ostomy seal.
- A sealing agent around fistula and wounds.
- An external packing agent on uneven skin.

Intended users

For ostomates, individuals managing fistulae or wounds, healthcare professionals and caregivers.

Instructions for use

Application

1. Remove the seal from its outer package.
2. Ensure that the area for application is clean, dry and free from traces of grease or cream.
3. Warm the seal in your hands for 20 – 30 seconds until soft before removing the protective discs. Gently stretch the eakin **Cohesive**® seal as required to fit the application site. It may also be cut and securely re-joined to fit around more difficult sites.
4. Apply to the skin with gentle pressure and hold in place until the eakin **Cohesive**® seal is secure on the skin. Mould the seal towards the base of the stoma or the edge of the wound. Do not allow the seal to cover over the top of open wounds or your stoma opening.
5. Ensure a snug, but not restrictive fit with no gaps around the base of the stoma, to provide maximum skin protection.
6. Place any appliance directly onto the surface of the eakin **Cohesive**® seal. Hold in place with gentle pressure until securely attached to the seal and the surrounding skin.

Removal

Gently remove any attached appliance from the body. Remove the eakin **Cohesive**® seal by carefully peeling and rolling from the outer edge. Wipe gently to ensure the skin is clean and dry. A medical adhesive remover can be used if necessary.

Disposal

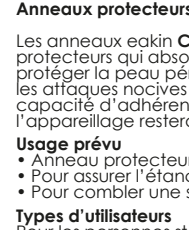
Dispose of waste products in accordance with local guidelines.

REF	≡	⊙	⊗
839001	2.1mm	18mm	98mm
839002	4.2mm	18mm	48mm
839005	2.8mm	18mm	48mm


en Outer Diameter **cs** Vnější průměr **de** Außendurchmesser **da** Udvendig diameter **es** Diámetro externo **fi** Ulkoalkaisija **fr** Diamètre extérieur **hr** Spoljni prečnik **is** Yfirir þvermál **it** Diametro esterno **nl** Buitendiameter **no** Ytre diameter **pl** Średnica zewnętrzna **pt** Diâmetro externo **sv** Ytterdiameter **sl** Zunanji premer

en Inner Diameter **cs** Vnitřní průměr **de** Innendurchmesser **da** Indvendig diameter **es** Sisálbimicót **fi** Sisäalkaisija **fr** Diamètre intérieur **hr** Unutarnji prečnik **is** Innri þvermál **it** Diametro interno **nl** Binnendiameter **no** Indre diameter **pl** Średnica wewnętrzna **pt** Diâmetro interno **sv** Innerdiameter **sl** Notranji premer

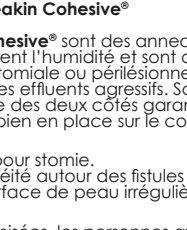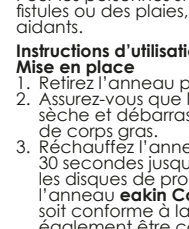
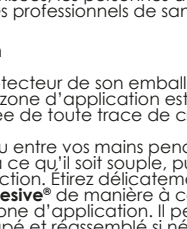
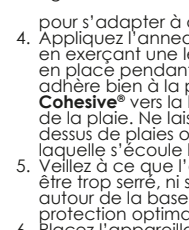
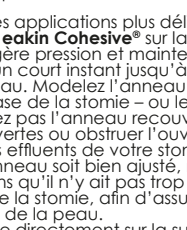
en Thickness **cs** Tloušťka **de** Stärke **da** Tykkelse **es** Espesor **fi** Paksuus **fr** Épaisseur **hr** Debljina **is** Þykkt **it** Spessore **nl** Dikte **no** Tykkelse **pl** Grubość **pt** Espessura **sv** Tjocklek **sl** Debelina

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